

dermalogica professional treatments in a medical setting

Dermalogica treatments with the Bio Therapeutic Ultimate Gold Microcurrent machine

When combining professional Dermalogica treatments and microcurrent with the Bio Therapeutic Ultimate Gold, it is important to:

1. Review the Consultation Card to provide an in-depth consultation of your patient/client's skin ensuring they are a candidate for treatment.
2. Ensure the skin is in optimum health to reduce possible complications.
3. Never leave your client unattended when using an electrical modality.
4. Use Dermalogica professional products that are suitable for the patient/client's current skin condition.
5. Directions and protocols may differ from machine to machine; always consult the manufacturer for exact protocols and product usage for your particular microcurrent machine. Microcurrent can be used for muscle re-education or specific iontophoresis.

Definitions with microcurrent therapy:

- **Preparation 1 and 2** – Muscle re-education
- **Skin 1, 2 and 3** – Specific Iontophoresis (product penetration)

Dermalogica treatment product recommendations based on skin condition for use with Bio Ultimate Gold Microcurrent therapy:

Premature aging skin:

- Cleanse with **PreCleanse** followed by **Skin Resurfacing Cleanser**
- Perform **Face Mapping® skin analysis**
- Exfoliate with **MultiVitamin Power Exfoliant** or **Exfoliant Accelerator 35**
- Remove with **Special Cleansing Gel**
- Prep 1 – **Multi-Active Toner**
- Prep 2 – **Antioxidant HydraMist**
- Skin 1 – **AGE Smart® Complex**
- Skin 2 – **MultiVitamin Power Recovery® Masque** (remove after Skin 2)
- Skin 3 – **Intensive Moisture Balance** or **Power Rich™**
- Apply **Dynamic Skin Recovery SPF30** or **Solar Defense Booster SPF50**

Hyperpigmentation:

- Cleanse with **PreCleanse** followed by **Tri-Active Cleanse**
- Perform **Face Mapping® skin analysis**
- Exfoliate with **Daily Microfoliant®** and **Exfoliant Accelerator 35**
- Prep 1 – **Multi-Active Toner**
- Prep 2 – **Antioxidant HydraMist**
- Skin 1 – **ChromaWhite TR_x® Complex** with **Purifying Botanical Mixer**
- Skin 2 – **MultiVitamin Power Recovery® Masque** (remove after Skin 2)
- Skin 3 – **Extreme C, C-12 Concentrate** and **Skin Smoothing Cream**

dermalogica®

dermalogica professional treatments in a medical setting

- Apply **Pure Light SPF30**

Sensitized skin:

- Cleanse with **PreCleanse** followed by **UltraCalming™ Cleanser**
- Perform **Face Mapping® skin analysis**
- Exfoliate with **Daily Microfoliant®**
- Prep 1 – **UltraCalming™ Mist**
- Prep 2 – **UltraCalming™ Mist**
- Skin 1 – **UltraCalming™ Complex**
- Skin 2 – **Colloidal Masque Base** (remove after Skin 2)
- Skin 3 – **UltraCalming™ Serum Concentrate**
- Apply **Ultra Sensitive Tint SPF30** or **Super Sensitive Shield SPF30**

Oily/acne skin:

- Cleanse with **PreCleanse** followed by **Dermal Clay Cleanser** or **Clearing Skin Wash**
- Perform **Face Mapping® skin analysis**
- Exfoliate with **Skin Exfoliant System**
- Prep 1 – **Multi-Active Toner**
- Prep 2 – **Multi-Active Toner**
- Skin 1 – **MediBac Clearing® Complex**
- Skin 2 – **Colloidal Masque Base** (remove after Skin 2)
- Skin 3 – **Active Moist**
- Apply **Oil Free Matte SPF 30** or **Solar Defense Booster SPF50**